# ProPig # 22655

- DE Bedienungsanweisung
- FR Mode d'emploi
- EN Instructions for use
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- PL Instrukcja obsługi
- DA Brugsanvisning
- ES Instrucciones de uso
- SV Bruksanvisning
- CS Návod k použití
- HU Használati útmutató
- NO Bruksanvisning
- SL Navodila za uporabo
- FI Käyttöohje
- HR Naputak za uporabu
- SK Návod na použitie
- PT Instruções de uso

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## 1. Vorwort

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## 1. Foreword

### 1.1. Information about the guide

We reserve the right to make changes due to further technical development compared to the data mentioned in this manual. Reprints, translations and reproductions in any form, including excerpts, require the written consent of the manufacturer.

## 2. Safety

### 2.1. General

This user guide includes instructions for use and safety instructions. Please read the instructions carefully and note the regulations and relevant information before putting the device into operation. Please keep this user guide in a safe place for later use.

### 2.2. Safety instructions

**CAUTION!** Danger of fire, short-circuits and electric shock!

- Do not pull on the electrical supply cable to unplug or transport the device.
- Do not allow electrical supply cables to come into contact with heat, oil or sharp edges.
- Store the device in a dry place and protect from liquids and moisture.

#### **Risk of physical injury and material damage if used incorrectly!**

- Ensure that the device is only used by people who have the professional skills required to do so.
- Note the maintenance instructions.
- Only operate the device at the mains voltage specified.
- Do not expose the battery pack to excessive temperatures, sunlight or fire.
- Avoid all skin contact and seal the device in a good plastic bag in the case of battery leakage.
- Protect the battery pack from vibration and shocks.
- Do not use the device if the casing is broken.
- Never press the locking button while the device is running.
- Keep out of the reach of children.
- National and international safety, health and occupational safety regulations must be observed and complied with.
- Keep your workspace clean and tidy. An untidy workspace increases the risk of accidents.
- Do not work without protective goggles.
- Do not work with damaged grinding wheels or bent shafts.
- In the case of overheating, switch the device off and allow to cool for several minutes.

### 2.3. Charging notes

- The tooth grinder is powered via a separate battery pack.
- Only charge the battery pack with the adapter provided with the original unit #22655.
- Remove the device from the battery for charging.
- Plug the transformer's device connector into the charging socket (IN) of the battery pack.
- Insert transformer into the socket. The charge control display on the transformer lights up red.
- Maintain constant supervision when charging. Do not leave the battery pack to charge unattended and do not leave the charger plugged in overnight.
- When the charging process is complete, the charge control display on the transformer lights up green.
- Stop charging after 3 hours. The charging process does not stop automatically and overloading leads to destruction and danger to people.
- Immediately interrupt charging if you notice the battery is heating up excessively.
- With a fully charged battery pack, the tooth grinder can be used for over 2.5 hours.

## 3. Description

### 3.1. Description of the device

The ProPig tooth grinder is for grinding down sharp edges on piglets' teeth.

### 3.2. Included with the device

- Tooth grinder
- Battery with belt hook
- Battery charger
- Power supply cable with Euro flat plug
- Power supply cable with UK plug
- Travel bag with belt for grinding machine
- Plastic carrying case
- Operating instructions

### 3.3. Correct use

The device should only be used for grinding down piglets' teeth within the framework of local animal welfare regulations. The regulations set out in these operating instructions must be complied with. Any other use of the device, e.g. in any other area than those described, shall be deemed incorrect use.

In the event of incorrect use or modifications to the device, the manufacturer's warranty and liabilities shall be deemed invalid. Improper maintenance, charging and cleaning will void the warranty. Furthermore, defects caused by too much pressure during use are excluded from the warranty.

### 3.4. Technical data

Handpiece 22655-0	Battery charger 22655-7
Supply voltage 18 V	AC input 100 – 240 V, 50/60 Hz
Speed in rpm 20.000	DC output 21 V, 2000 mA
Power consumption 40 watts	Protection class II
Protection class III	

## 4. Commissioning and operation

### 4.1. Included with the device

Supplied in a storage box.

### 4.2. Review

Check the delivery for completeness and damage.

### 4.3. Commissioning

- The battery pack should be charged before initial operation.
- Familiarise yourself with the device before starting the grinding process on teeth.
- Only connect the device to the battery pack once switched off.
- Put on the carry strap. You can easily affix the battery pack to the strap using the clip. The tooth grinder can then be placed in the bag provided for this purpose.
- Plug the tooth grinder's device connector into the socket (OUT) of the battery pack.
- Switch on the device using the on/off switch.
- Switch off the device after use.
- Always switch off the device if the grinding process is interrupted.

### 4.4. Operation

- Ensure that the grinding tool is cleaned, see "Cleaning and Care". This is the only way to achieve effective grinding performance.
- Do not forcefully press the grinding tool against the teeth. This reduces the grinding effect and the grinding tool is stressed unnecessarily.
- Only grind the tooth at the tip using light pressure to remove the sharp edges.
- Ensure that you only grind the tips of the teeth. If you remove too much tooth enamel, you will damage the pulp area and give pathogens an entryway.

## 5. Maintenance and cleaning

### 5.1. Cleaning and care

Clean the grinding tool after each use. Remove the grinding tool and aluminium protective cap from the device (see below for procedure) and clean it with a hard brush and water or vinegar. Thorough cleaning of both parts can also be carried out using a commercially-available cold cleaner. After cleaning, grease the clamping sleeve and the socket of the union nut (external thread).

### 5.2. Maintenance

Check all components for signs of damage before every use. Damaged parts should only be repaired by a qualified professional or the manufacturer. If carrying out any repairs, only use original spare parts.

#### Removing and changing the grinding tool

Only use a genuine grinding tool (item number 22913) as a replacement grinding tool in conjunction with the pre-assembled clamping sleeve (ø 3.0 mm).

Proceed as follows:

Disconnect the mains plug before changing the tool. Never press the locking button while the device is running.

1. Remove the aluminium protective cap.
2. Slightly turn the union nut and press the locking button at the same time until it clicks.
3. Open the collet chuck and swap out the tool. Clamp all tools as short as possible. Long protruding shafts bend easily and cause poor running.
4. Replace the aluminium protective cap.

Caution! The grindstone must not protrude beyond the aluminium protection cap. If this is the case, the grindstone must be released again and the grinding pin must be moved further into the unit and re-clamped.

## 6. Spare parts

22878 Aluminium protective cap

22913 Grinding tool

22658 Battery

## 7. CE-/UKCA-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE-/UKCA-mark confirms compliance with the Directives of the European Union or the relevant UK legislation.

## 8. Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

## 9. Guarantee

Albert Kerbl GmbH grants you an XX-year guarantee on this article in accordance with our general guarantee conditions. These general warranty conditions can be found at [www.kerbl.com/guarantee](http://www.kerbl.com/guarantee). In addition to these general warranty conditions, the following exclusions also apply: Wear and rubber parts. Your statutory warranty rights remain unaffected by our guarantee.

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## **Kerbl Austria Handels GmbH**
## **Kerbl UK Ltd**